Dispenser

Tipo 2: para cartuchos 1:1 ó 1:2 (50 ml)
 Tipo 3: para cartuchos 1:1 ó 1:2 (25 ml)

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




Instructions for use

Dispenser for use of dental materials in cartridges (see above).

Application:

1. To insert the cartridge, release the lock (A) and pull back the feeder (B) completely. To do so, push the lever (C) up and pull the feeder back. (Fig. 1)
2. Insert the cartridge and engage the lock. Check that the cartridge is inserted in the **Dispenser** correctly: The recesses on the cartridge and **Dispenser** should line up. (Fig. 2)
3. Turn the cartridge seal (D) or any mixing tip you have used 90° and remove. (Fig. 3)
 Due to technical reasons slight differences in the filling level between the cartridge chambers can occur prior to the first use. For this reason, squeeze the trigger lever (E) to express material until the material exits both outlets evenly. (Fig. 3a)
4. Connect a new mixing tip to the cartridge by turning it 90°. Check that the mixing tip is attached to the cartridge correctly: The recesses on the cartridge and mixing tip should line up. (Fig. 4)
5. Squeeze the trigger lever (E) to mix the material in the mixing tip and apply. After use leave the mixing tip on the cartridge or re-attach the cartridge cap. (Fig. 5)
6. To remove the cartridge, press the lever (C) on the rear of the **Dispenser** up, pull back the feeder (B) completely and release the lock (A).
7. After use, clean and disinfect the **Dispenser** by hand. For the disinfection, use conventionally available disinfectants based on peroxides, combinations of aldehydes/ alcohols, phenols, alkaline alcohol-based solutions or guanidines, glycol derivatives, quaternary compounds and alkylamines specifically approved for the disinfection of instruments on the VAH list (<https://vah-liste.mhp-verlag.de>). The specifications of the disinfectant manufacturer must be observed and followed.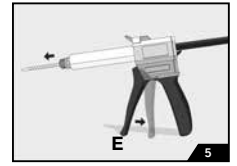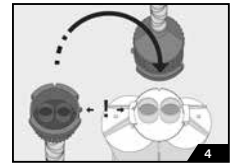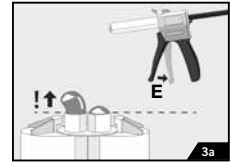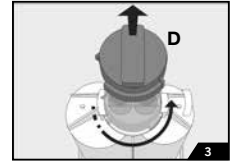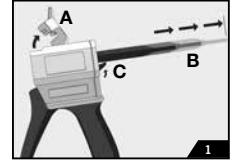

Additional Information:

The **Dispenser** (type 2, type 3) allows rapid and uncomplicated application. The mixing tips ensure that the material is applied homogeneously in the correct ratio. The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**. Do not use visibly damaged devices.

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