# Futurabond® DC

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## Aplicación directa

The diagram illustrates the direct application of Futurabond DC cement in five steps:

- Step 1:** Shows the application of the cement to the tip of a dental instrument.
- Step 2:** Shows the cement being applied to the tooth surface. A '3x' magnification box highlights the application point.
- Step 3:** Shows the cement being applied to the tooth surface for 20 seconds.
- Step 4:** Shows the cement being applied to the tooth surface for 5 seconds.
- Step 5:** Shows the cement being applied to the tooth surface for 10 seconds.

Additional information: A box with the text "¡SIMPLEMENTE PRESIONAR! ¡SIMPLEMENTE ADHERIR!" (Simply press! Simply adhere!) is shown next to the product packaging.

## Aplicación indirecta

The diagram illustrates the indirect application of Futurabond DC cement in two rows:

- Top Row (Cementación):** Shows the application of the cement to the tooth surface for 5 seconds (Step 4) and 10 seconds (Step 5). An arrow points to a box labeled "Cementación".
- Bottom Row (Postes):** Shows the application of the cement to the tooth surface for 5 seconds (Step 4) and 10 seconds (Step 5). An arrow points to a box labeled "Postes".

Additional information: A box labeled "Instructions DC Cement" is shown next to the top row. A red 'X' is placed over the bottom row, indicating that the cement is not recommended for posts.

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VOCO GmbH