AIR-FLOW® CLASSIC

Before using this product, please carefully read and follow the recommendations of the instruction manuals originally delivered with your device and system. Please pay special attention to the safety precautions.

Product Description

The AIR-FLOW® Classic (prophylaxis powder) is used solely for supragingival treatments.

AIR-FLOW® Master and AIR-FLOW® Master Piezon users: this powder has to be used only with the AIR-FLOW® or the AIR-FLOW® CLASSIC+SOFT powder chamber.

This powder is for use in dentistry with the EMS air polishing devices.

Applications

This product is recommended for:

- Removing stains/colorations and dental plaque,
Plaque removal for placement of sealants,
Surface preparation prior to bonding/cementation of inlays onlays, crowns and veneers,
Preparation des surfaces avant la pose d'inlays, de couronnes ou de facettes,
Préparation des surfaces avant restauration avec des composites,
Nettoyage avant la pose de brackets,
Cleaning prior to bonding orthodontic brackets,
Effective plaque and stain removal for orthodontic patients,
Clean implant fixture prior to loading,
Stain removal for shade determination,
Plaque removal prior to fluoride treatment,
Plaque and stain removal prior to whitening procedure.

Contra-indications

Patients suffering from upper respiratory tract infections, from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

Patients on a low salt diet (for example hypertension) must not be treated with the powder as it contains sodium bicarbonate. For these patients it is recommended to use the salt free powders also provided by EMS (see below).

The flavor contained in the powder can lead, in specific cases, to allergic reactions. For patients known to suffer from such reactions, use the "Neutral" powder provided by EMS.

Do not direct the jet of powder toward fillings, crown or bridgework as this could damage these restorations.

Do not use this powder subgingivally. For this application use only the powder recommended by EMS for subgingival application.

Recommendations for the treatment

- Patient and user have to wear eye protection.
User has to wear a mask and gloves.

Use the high-speed evacuator of your dental unit.

Apply Vaseline® on the patient's lips.

General information

- Do not exceed the powder filling limit as indicated on the powder chamber.
This powder is sugar free.
If the bottle is left open, there is a risk of agglomeration of the powder, that could block the device.

Storage and shelf-life

- The powder should not be stored near acids or heat sources.
Storage condition up to 40°C and 10% to 90% relative humidity.
Transport conditions -29°C to 38°C and 10% to 90% relative humidity
Do not use after expiration date.

Also available:

- AIR-FLOW® CLASSIC available in Lemon, Mint, Cassis, Cherry, Tropical and Neutral (for supragingival treatments).
AIR-FLOW® CLASSIC COMFORT (for supragingival treatments).
AIR-FLOW® SOFT (salt free for supragingival treatments).
AIR-FLOW® PERIO (salt free for subgingival application).
AIR-FLOW® PLUS (salt free for subgingival and supragingival treatments).

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