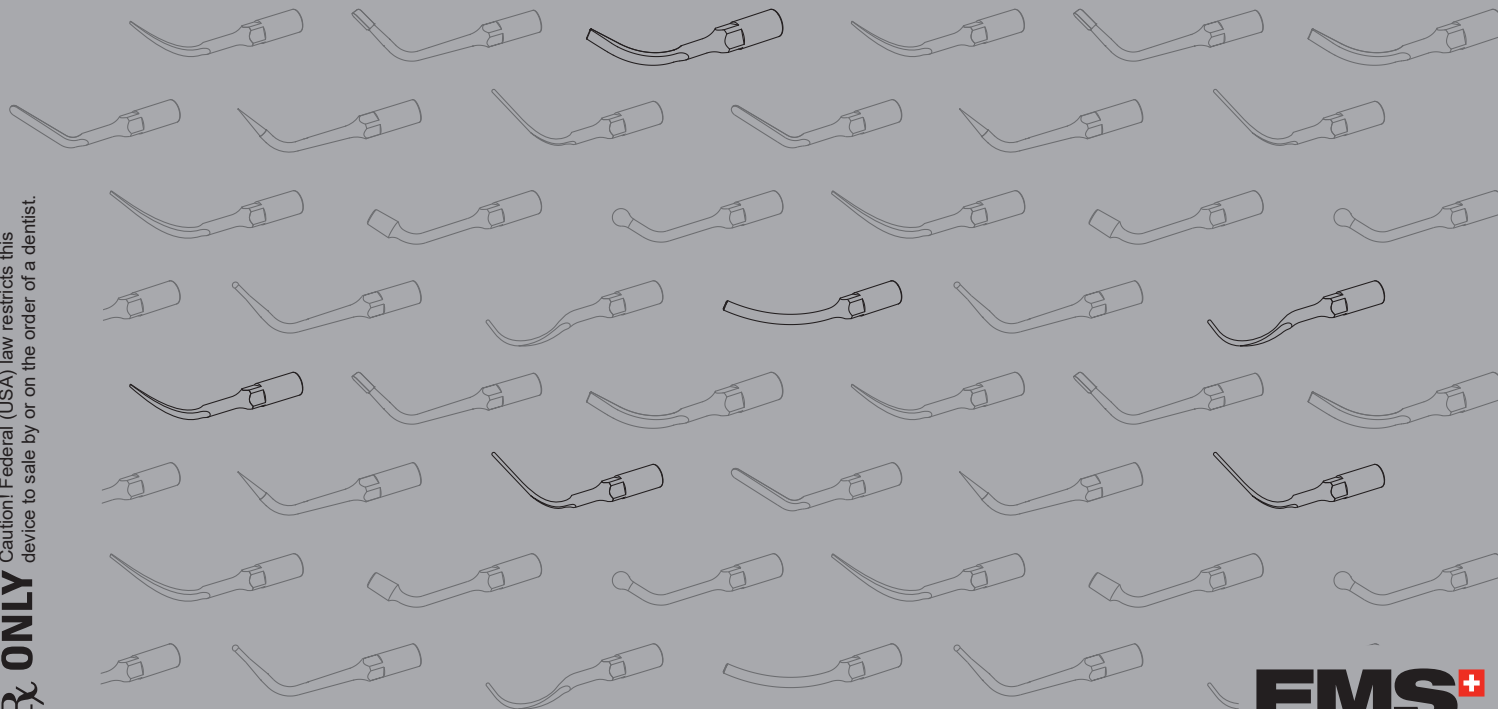
OPERATION INSTRUCTIONS  
 MODE D'EMPLOI  
 BEDIENUNGSANLEITUNG  
 INSTRUCCIONES DE EMPLEO  
 ISTRUZIONI PER L'USO  
 BRUKSANVISNING







# Piezon® Systems
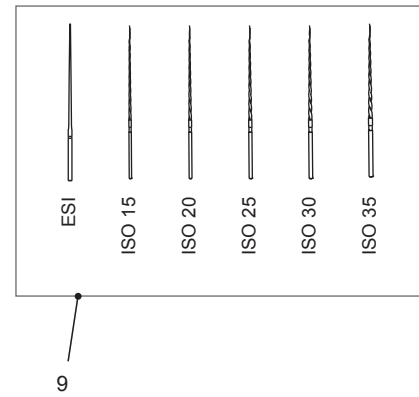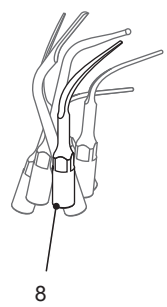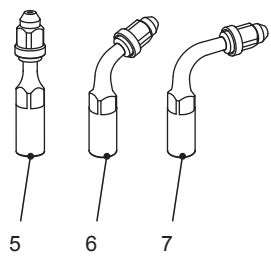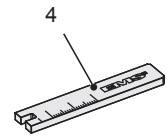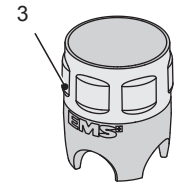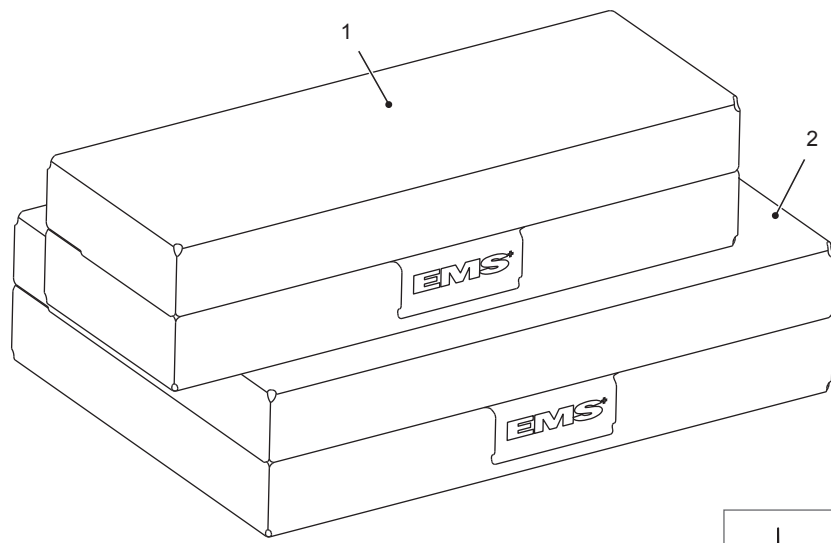
## PIEZON CLASSIC INSTRUMENTS

**Rx ONLY** Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

## COMPONENTS

- 1 SystemBox ¼ DIN
- 2 SystemBox ½ DIN
- 3 CombiTorque®
- 4 Flat wrench for file holders
- 5 File holder 180°
- 6 File holder 120°
- 7 File holder 90°
- 8 Instruments (typical shown)
- 9 K-type and ESI files
- 10 File container

The packing list shows what instruments and accessories are included in this system. To complete this set, instruments and accessories may be ordered individually or by set through your EMS dealer.

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**Endo files**

## DEAR CUSTOMERS,

Thank you for buying a new EMS Piezon system. It meets the highest quality and safety standards.

### Indications

EMS offers a wide range of Piezon ultrasound instruments for use in dentistry with an EMS Piezon ultrasound unit. Long thin instruments allow for easy access to periodontal pockets and interproximal areas. Piezon instruments are available for the following indications:

#### Scaling

- Removal of supragingival calculus
- Removal of stains

#### Endo

- Preparation, cleaning and irrigation of root canals
- Retrograde preparation of root canals
- Condensing gutta-percha
- Removal of crowns, bridges and restorations

#### Restorative

- Preparation of cavities
- Cementation of restorations
- Condensing of amalgams

#### Periodontics

- Scaling and root planing
- Periodontal therapy

### Contraindications

 EMS recommends not to treat patients with a cardiac pacemaker or a defibrillator with this product. The functionality of these devices may be affected by the high frequencies of the ultrasonic oscillations.

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## PIEZON CLASSIC

 The illustrations in these instructions represent generic components; however these instructions apply to the Piezon CLASSIC range of instruments (unless otherwise specified).

 The markings on the instruments indicate their type as listed in this manual.

 These ranges of Piezon instruments must not be used with an EMS Piezon Master Surgery® unit.

 The EMS Piezon family includes several specific CombiTorque. It is mandatory to always use the correct CombiTorque with the corresponding instruments and handpiece. The CombiTorque are easily identified by their body color as follows:

- Yellow CombiTorque .....Piezon CLASSIC
- Blue CombiTorque.....Piezon SURGERY

 Make sure that only compatible components are used for installation and removal of the instruments to avoid damaging this equipment.

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## ABOUT THIS MANUAL

These operating instructions are to ensure the correct installation and use of this product.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this document close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

### **Danger**

Risk of patient or user injury

### **Caution**

Risk of damage to the device or environmental harm

### **Please note**

Useful additional information and hints

### **Prohibited**

### **Authorized**

Instructions concerning the components of the system are provided separately in the relative instructions for use. Read the instruction manuals delivered with each of the components before using this product. Do not hesitate to contact your local EMS dealer for further information.

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Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call your approved EMS repair center or dealer directly.

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Ihre EMS

## Technical modifications

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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## CONTENTS

General hints.....	2
Bacteria reduction in the spray .....	4
Attaching the instruments.....	6
File holders.....	8
Positioning files or instruments.....	10
Working technique .....	14
Diamond-coated instruments.....	16
Scaling .....	18
Instrument A for universal scaling .....	21
Instrument B for lingual scaling.....	23
Instrument perio P.....	25
Instrument perio slim PS.....	27
Instruments PSR and PSL.....	29
Specific instrument PI for implants cleaning .....	31
Endo .....	32
Files for the orthograde therapy .....	32
Endodontic retreatment.....	34
Retrograde endodontics .....	36
K-type cleaning files (ISO 15 - 35).....	39
Endo soft instrument ESI (ISO 15).....	41
Retreatment instrument RT1.....	43
Retreatment instrument RT2.....	45
Retreatment instrument RT3.....	47
Condensation instrument H .....	49
Removal instrument D.....	51
Retrograde instrument Berutti .....	53
Retrograde instrument for isthmus RE2 .....	55
Restorative.....	56
Cavity preparation.....	56
Amalgams and cementation .....	58
Cavity instruments SD and SM.....	61

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Small ball instrument SB.....	63
Small ball instruments SBd and SBm.....	65
Instrument for pits and fissures PF.....	67
Instrument for veneers VE.....	69
Cementation instrument SP.....	71
Periodontics.....	72
preio line.....	74
Working with curved instruments.....	75
Perio line instruments PL1 and PL2.....	77
Perio line instrument PL3.....	79
Perio line instruments PL4 and PL5.....	81
Instruments perio HPL3 and DPL3.....	83
Cleaning, disinfecting and sterilizing.....	84
Instrument disassembly and cleaning.....	86
Handpiece disassembly and cleaning.....	88
File holder disassembly and cleaning.....	90
Wear and maintenance.....	92
Safety precautions.....	94
Storing the product.....	98
Product disposal.....	100
Warranty.....	100
Accessories.....	100
EMS customer service.....	102
Symbols.....	104
Technical data.....	105
Troubleshooting.....	108

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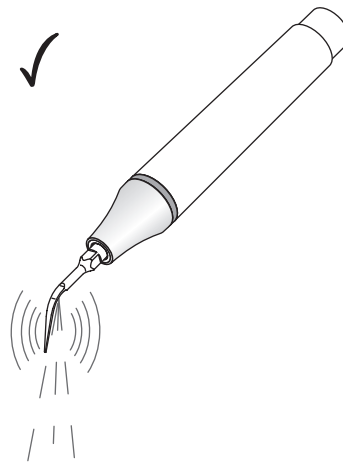
## GENERAL HINTS

## RECOMMANDATIONS

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 Make sure to always sterilize handpieces and instruments before use. Non-sterile handpieces and instruments may cause bacterial or viral infections. Please conform to the recommendations of the "Reprocessing Instructions" manual delivered with your product regarding procedure of cleaning, disinfecting and sterilizing of the components.

 Never work dry with Piezon instruments (unless otherwise specified). The tip of the instrument immediately heats up when used dry. Always make sure there is enough irrigation.

 Do not use the instruments on metal or ceramic restorations, neither on prosthesis units as they might be damaged (unless otherwise specified).

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Oscillating or rotating dental instruments produce a spray of irrigation. The bacterial count in the spray can be truly reduced by using the ready-for-use BacterX<sup>®</sup> pro\* solution (0.2% chlorhexidine) in the EMS Piezon unit.

 Rinsing the patient's mouth prior to treatment with BacterX<sup>®</sup> pro\* is strongly recommended.

Used daily in your EMS Piezon unit, BacterX<sup>®</sup> pro\* reduces the bacterial count in the irrigation lines of the unit.

\* BacterX<sup>®</sup> pro is available in specific countries only

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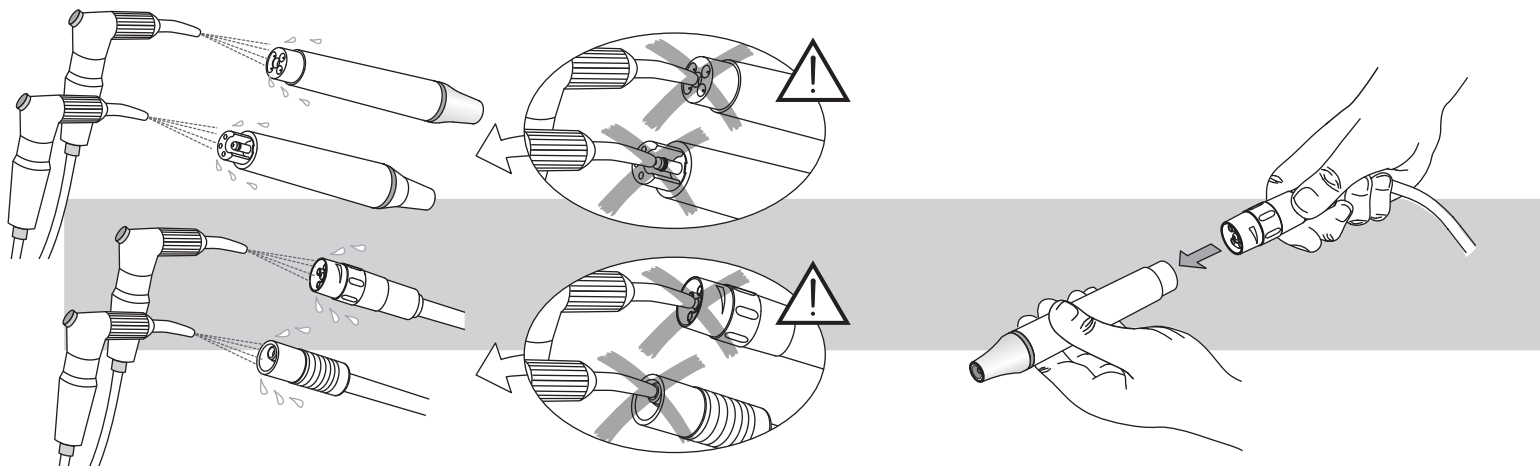
## FIXATION DES INSTRUMENTS

## BEFESTIGEN DER INSTRUMENTE

**!** Blow dry the connections to remove eventual presence of liquid to ensure a proper electrical contact. Never blow compressed air in the irrigation connections as this will irretrievably damage internal parts.

**!** Only use the CombiTorque to tighten the EMS instrument on the handpiece to the correct torque. Once the instrument is screwed all the way in, give an extra quarter of a turn to obtain the required torque.

**!** The EMS CombiTorque combines a torque wrench and an individual file holder. It ensures an installation according to specifications, an organized storage and a safeguard against injuries or contamination.

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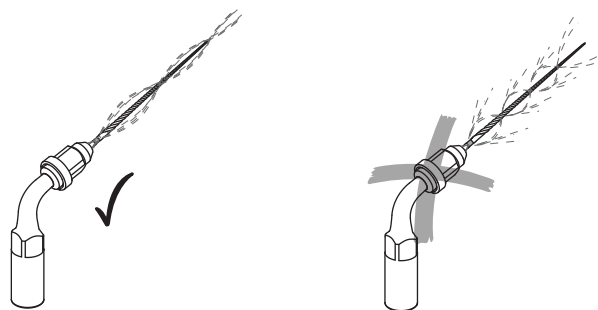
**!** The spray pattern of irrigation liquid will not be appropriate if the file or the instrument is not properly positioned in the file holder. The file may break, rub the walls of the root canal and enlarge the apex orifice. It is therefore important to verify that this positioning is correctly set.

**!** Always use the handpiece with the nose cap installed and check that it is correctly tightened.

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


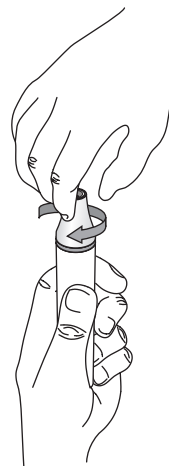
## WORKING TECHNIQUE

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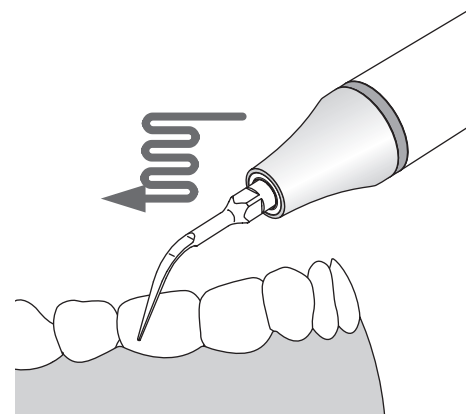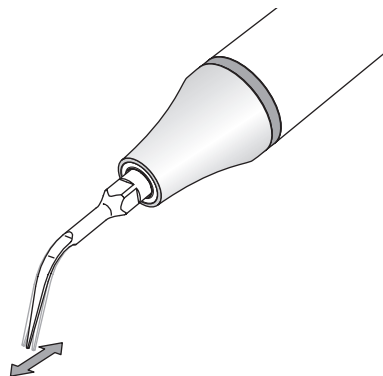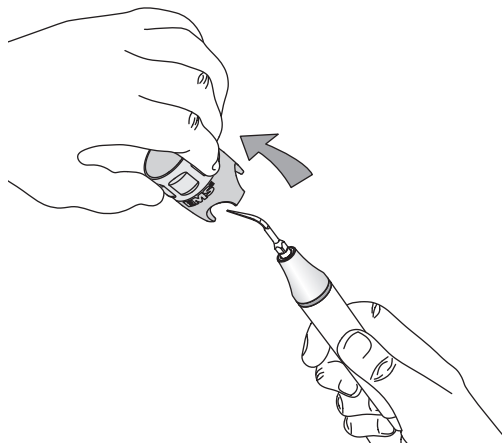EMS Piezon instruments vibrate in a controlled back and forth oscillation. During treatment, always hold the instrument tangentially to the tooth surface. Move the handpiece similarly to a paintbrush while applying a light lateral pressure.

For a given power setting, long and thin instruments deliver less clinical power, meaning a more gentle treatment for the patient. Shorter instruments can be selected when a higher clinical power is needed.

 Do not direct the instrument straight to the enamel surface. Never point the tip of the instrument to the tooth surface (unless otherwise specified).

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
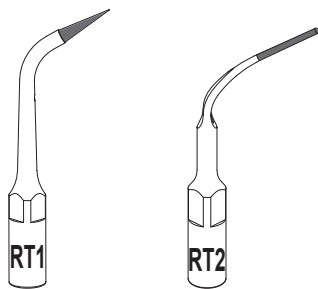
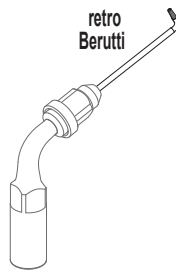
## Instruments diamantés
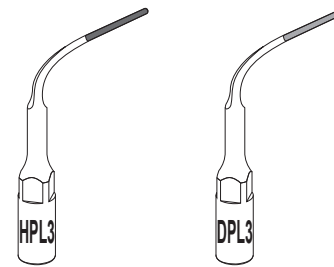
## Diamantierte Instrumente

 The diamond-coated instruments are very efficient. Always use them with sufficient irrigation flow to avoid damaging hard and soft tissues and reduce wear.

 The ultrasound vibrations cannot be delivered properly when the tip of the instrument is under heavy load. A gentle pressure on the instrument will free its full effective power without risk for the tissues, and enable a noteworthy wear reduction.

 Visually inspect the diamond coating before use. A worn coating greatly reduces the efficiency.

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## SCALING

 Avoid using the HVE (high volume evacuation) to ensure sufficient cooling of the tooth and the instrument. The instrument heats up quickly if the irrigation flow rate is not enough.

### Instrument A

Universal scaling instrument for removing supragingival deposits in all quadrants.

### Instrument B

Instrument for removing large supragingival deposit in all quadrants. It can be used for removing remains of orthodontic cements.

### Instrument P

Instrument for removing supra or subgingival deposits in all quadrants, particularly in the interproximal and the sulcus areas.

### Instrument PS

Slim instrument for subgingival fine scaling and for root debridement.

### Instrument PSR and PSL

Instruments PSR (left-curved) and PSL (right-curved) are used for debridement and provide easy access to interproximal areas.

## Implants and restorations cleaning

### Instrument PI (on file holder)

Specific instrument for removing plaque and calculus from the surface of implants and from metal or ceramic restorations.

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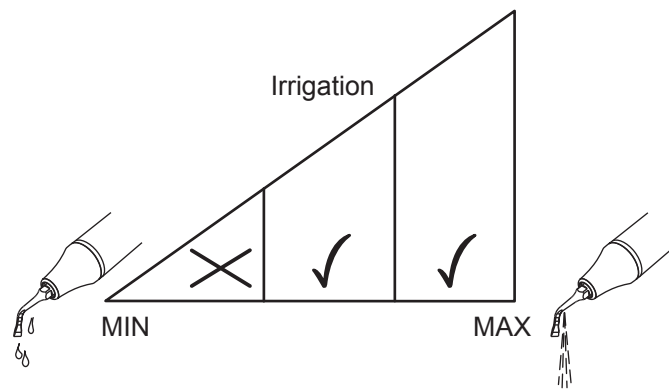
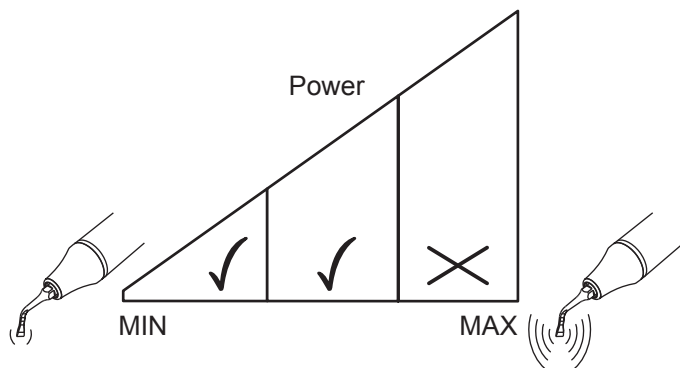
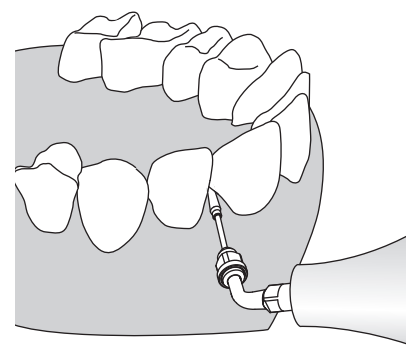
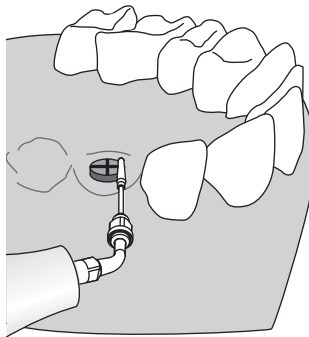
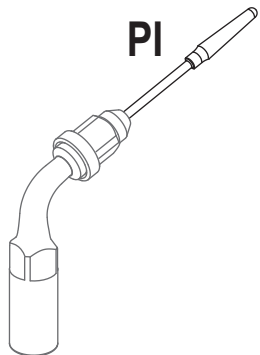
## Specific instrument PI for implants cleaning

Removal of plaque and calculus from the surface of implants and from metal or ceramic restorations. Surface cleaning on prosthesis units. Must be used with a 120° file holder.

Use a low power setting or medium power for hard concretions.

Use a high to medium irrigation flow rate.

 Verify that the plastic coating is not worn or damaged before use. We recommend to use a rubber dam to prevent the patient from accidentally swallowing or inhaling debris. When a rubber dam cannot be used, instruct the patient to breathe through the nose.

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### Files for the orthograde therapy

#### K-type files (on file holder)

Stainless steel endo files activated by high frequencies for shaping, cleaning and disinfecting the root canal system. Available in sizes ISO 15 to ISO 35, they can be used with a 90° or a 120° file holder.

#### ESI file (on file holder)

NiTi file in size ISO 15 for cleaning and irrigation of the root canal system. A 120° or a 90° file holder can be used.

 Never activate the files without irrigation and never operate them outside the root canal otherwise they will break (except for 2 sec. during the setting of the irrigation).

 To prevent file from breaking, create a glide path with a hand endo file and plan for an access as straight as possible to limit file twisting. Use a light touch-retract motion without rush. Often examine the file for detection of signs of fatigue. In application of the precautionary principle, replace the file as soon as possible.

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ENDO

## Endodontic retreatment

The instruments allow for efficient retreatment of the root canals. They are used to gain access to broken instruments or files inside a root canal and to remove them. They are also used for removal of crowns.

### Instrument RT1

Conically-shaped and diamond-coated instrument for the location of root canal entry and removal of calcifications in the first coronal third of the root canal system.

### Instrument RT2

Thin diamond-coated instrument for removing shoulders and other obstacles as well as for creating a straight canal to access a broken file.

### Instruments RT3 (on file holder)

Long and thin instruments for removing of broken file parts from the root canal.

 Use the ESI file to finish the root canal following the removal of the broken file.

### Instrument H

Long instrument for the lateral condensation of gutta-percha in the root canals. The heat generated by the high frequencies on the instrument tip softens the gutta-percha and maximizes lateral flow through the root canal system.

### Instrument D

Instrument for transmitting ultrasonic vibrations to loosen the cement that attach crowns and bridges to the tooth.

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## Retrograde endodontics

The retro instruments are used for the retrograde preparation of root canals. They are designed for minimally invasive and efficient treatment.

### Retrograde instrument Berutti (on file holder)

Diamond-coated instrument for retrograde preparation of the root canal allowing for cutting inside the tooth structure with a low power setting. A 120° or a 180° file holder can be used.

### Retrograde instrument RE2 (120° or 180° file holder)

Instrument thinner than an instrument Berutti for the retrograde preparation of isthmus between the root canals. The surface of the tip is titanium nitride coated for optimal abrasion and precise cutting.

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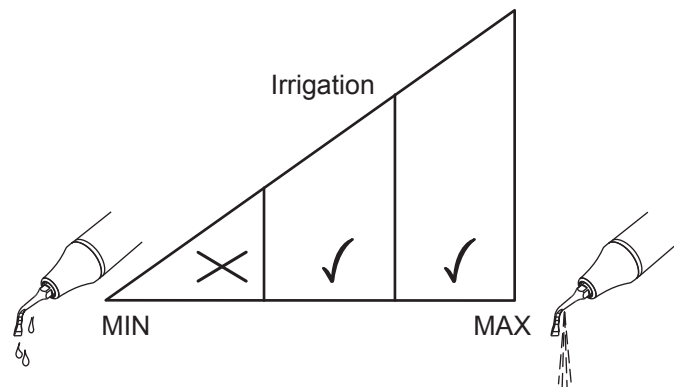
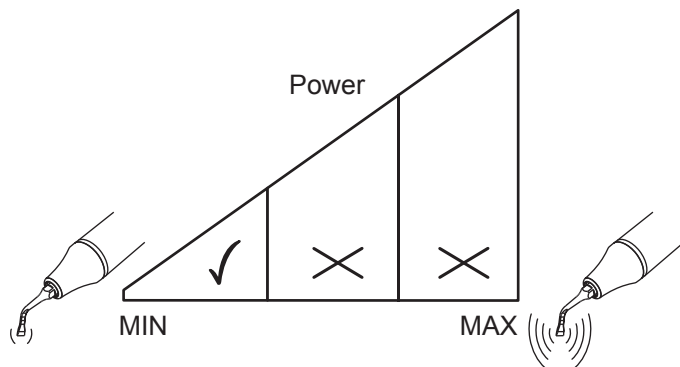
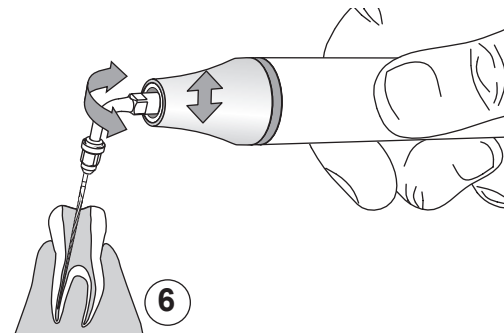
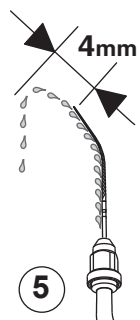
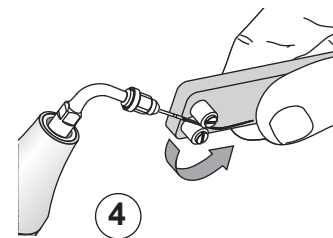
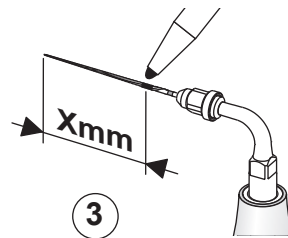
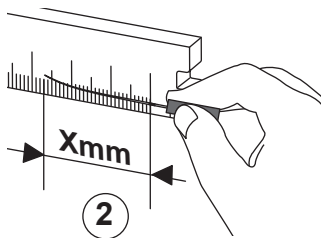
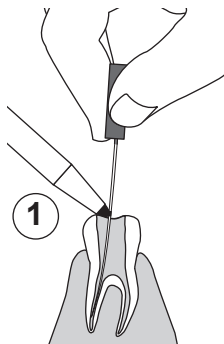
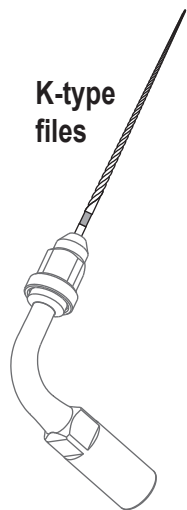
K-type files

## K-type cleaning files (ISO 15 - 35)

Files made of stainless steel for the preparation, cleaning and disinfection of the root canal system. Must be used with a file holder.

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 Activate the file for 4 seconds. Do not exceed 10 continuous seconds.

 To set the flow rate, activate both the irrigation and the ultrasound with the tip pointing upwards. The spray must extend by 4 mm the file tip.

 The root canal must be prepared using hand files in size ISO 15 before using the K-type files. Preform the file to the canal morphology if needed. Use slow circular and up-and-down movements and use the "step-back" technique.

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

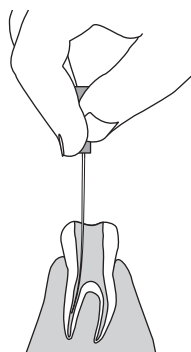
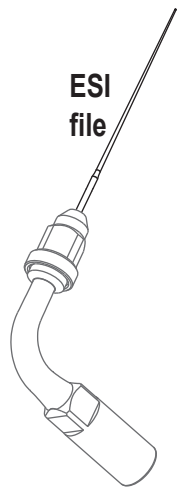
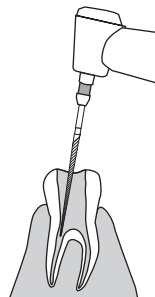
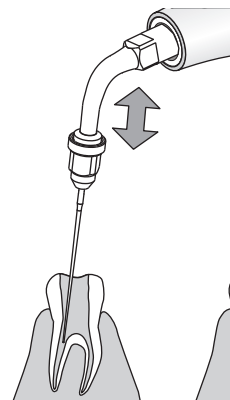
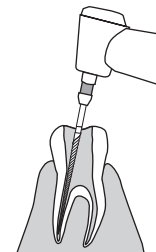
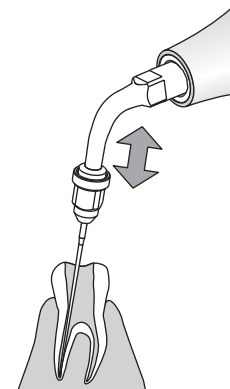
5
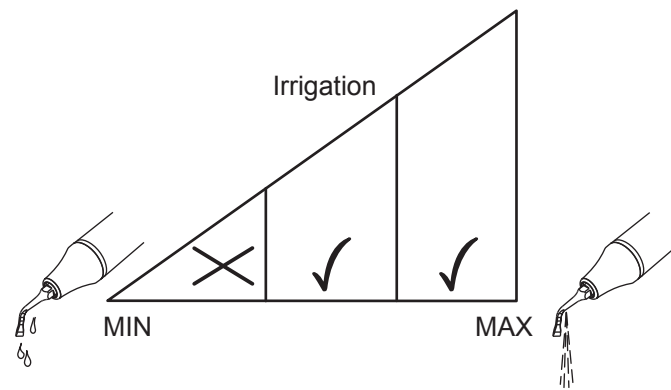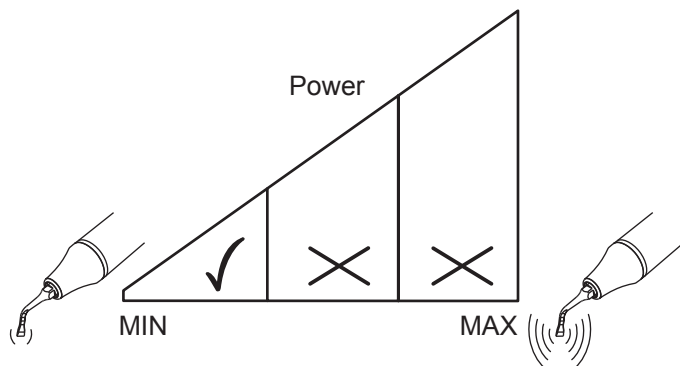
## Endo soft instrument ESI (ISO 15)

Smooth non-cutting file made of nickel titanium alloy (NiTi) for cleaning and irrigation of the root canal system. Must be used with a file holder.

Use Endo mode (if available) with a low power setting (maximum 30%), and use a medium to high irrigation flow rate.

 The ESI files can be used for a maximum of fifteen straight root canals, but for one single root canal when it is curved.

 The root canal must be prepared using hand files in size ISO 15 before using the ESI file. The ESI file will be used in sequence with a rotating or manual shaping system.

 Activate the ESI file for an approximate maximum of 3 x 20 seconds with high irrigation for each canal. Use gentle up-and-down movements to irrigate the root canal.

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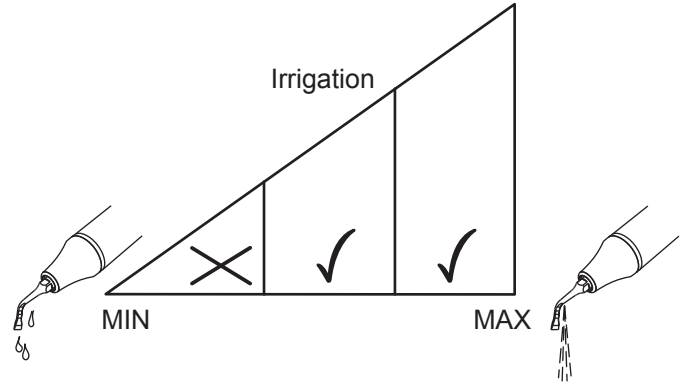
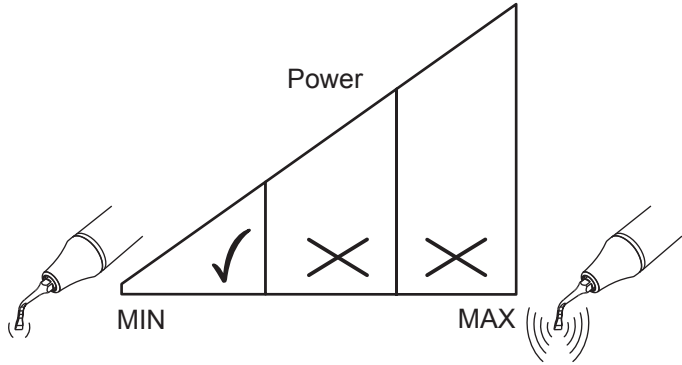
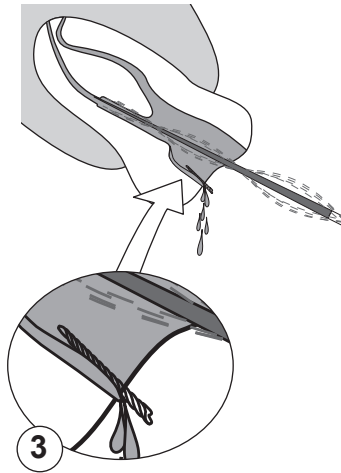
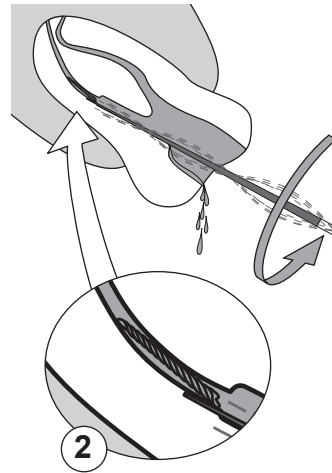
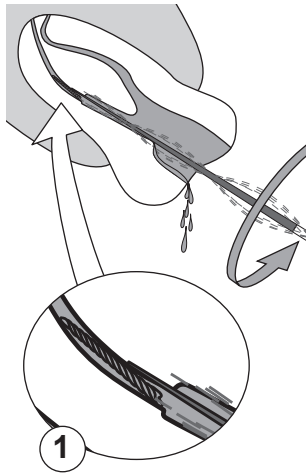
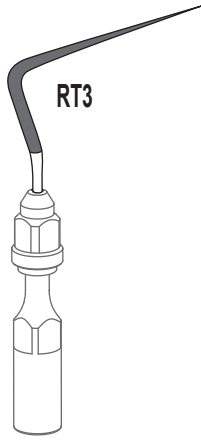
### Retreatment instrument RT3

Instrument for the removal of files broken inside a root canal. Must be used with a file holder.

Use a low power setting

Use a medium to high flow rate.

If possible place the patient head in a position where the root canal is horizontal with a downward inclination. Rotate the tip of the instrument counter-clockwise around the broken part until it is free from the root canal.

The combined effect of irrigation flow and gravity enables the extraction of the broken file and debris from the root canal.

 Avoid contact between instrument and broken file to not push it deeper. Do not apply pressure to the instrument in the axial direction.

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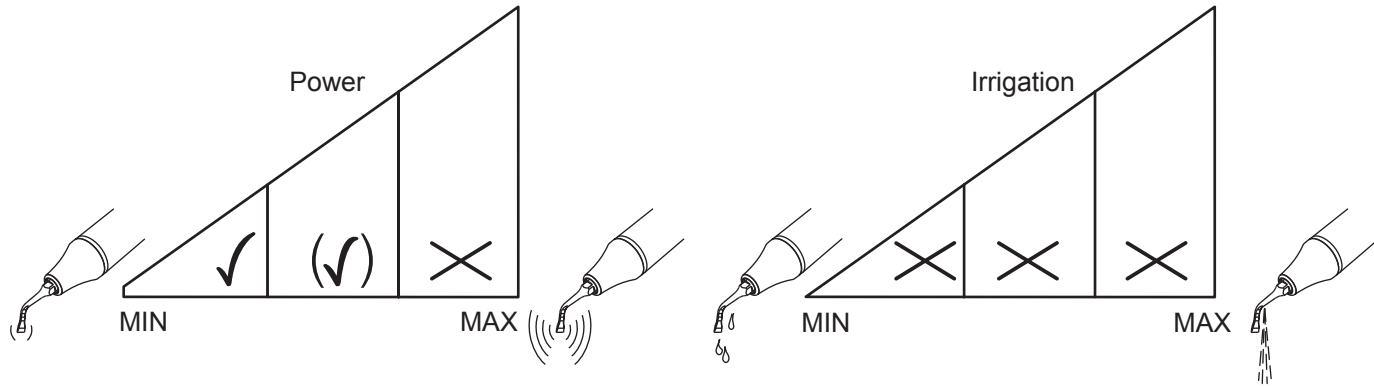
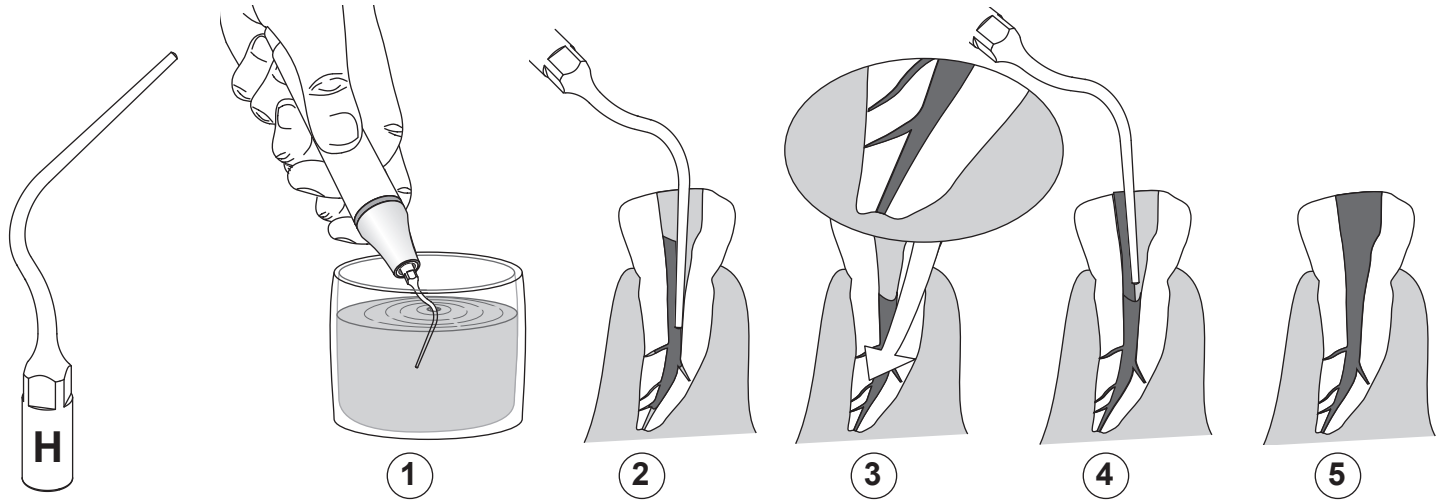
## Condensation instrument H

Thermal condensation of gutta-percha.

Use a low to medium power setting and no irrigation.

The heat generated by the ultrasound at the tip of the instrument liquefies the gutta-percha, thus irrigation is not wanted due to its cooling effect.

 Immerse the instrument in eugenol before inserting it in the canal. Slightly press the instrument against the cone of gutta-percha and activate the ultrasound for 2 seconds to condense. Repeat for each inserted cone until the canal is filled.

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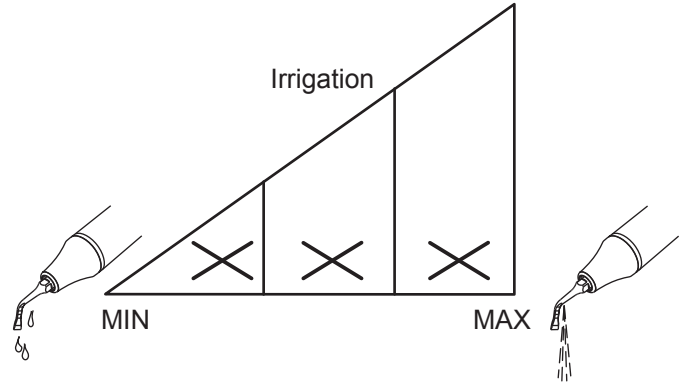
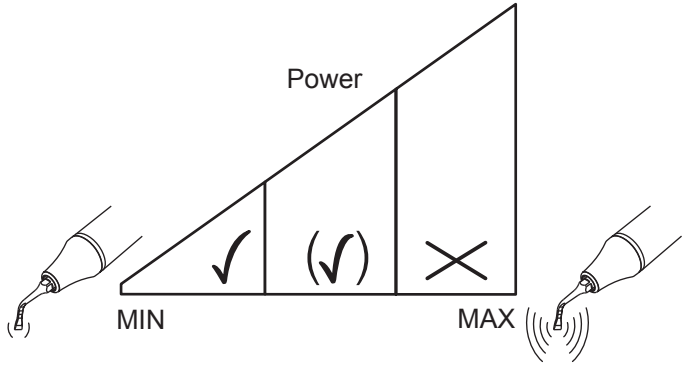
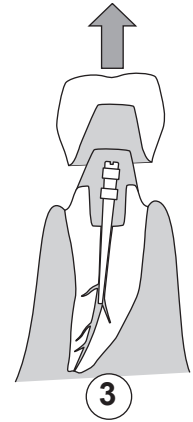
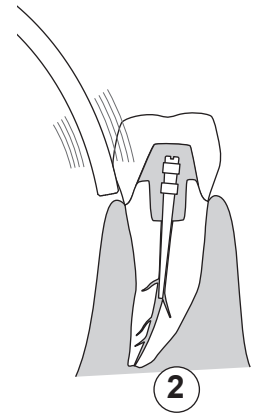
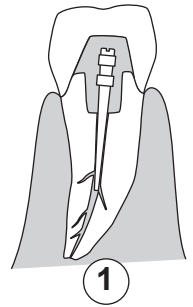
## Removal instrument D

Removal of crowns and bridges (works on cements with mechanical or micro-mechanical retention principle; it does not work on adhesive bonded cements).

Use a low power setting and increase as needed. Do not use irrigation.

Apply the instrument onto surface and activate. Increase pressure until vibrations can no longer be heard and maintain for few seconds. An intermittent activation avoids injuring the pulp with heat by allowing time for heat dissipation.

 The instrument D is useful for loosening cementation on hard materials. However the high frequencies of ultrasound applied onto ceramics or composites may eventually fracture them.

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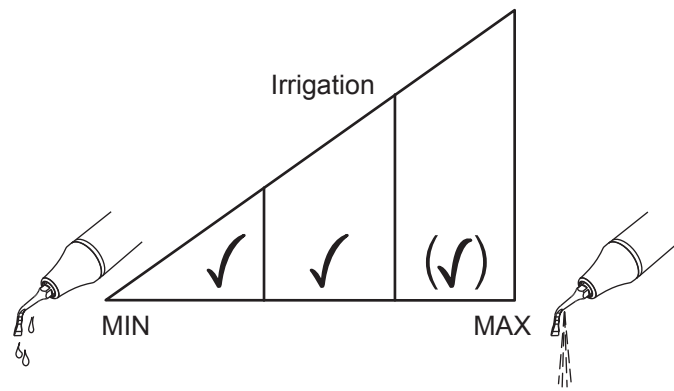
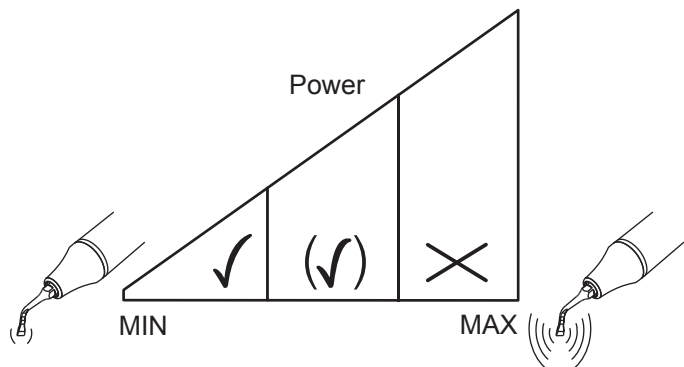
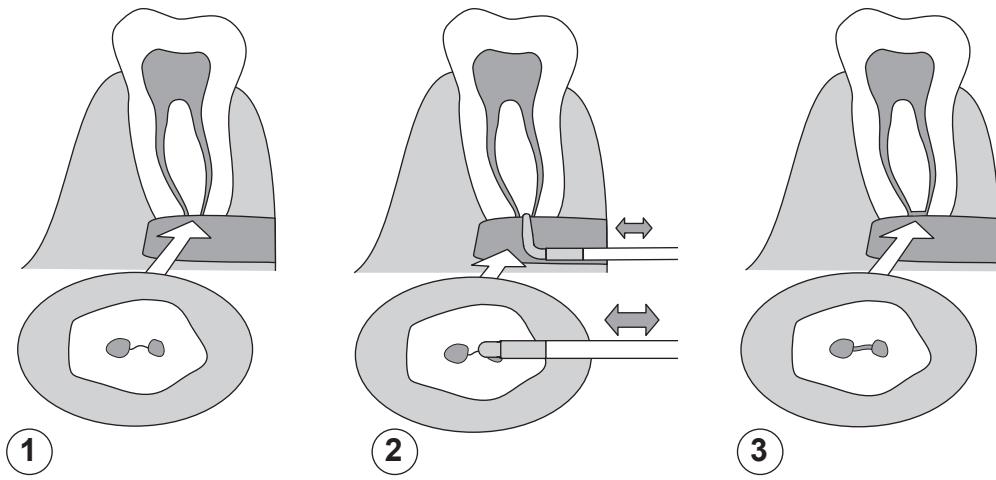
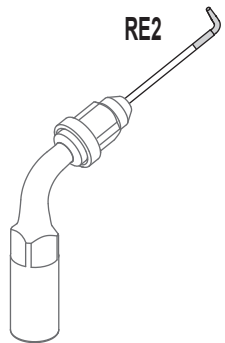
## Retrograde instrument for isthmus RE2

Titanium nitride coated instrument for the retrograde preparation of the isthmus between the root canals. Must be used with a file holder.

Use a low to medium power setting.

Use a low to high irrigation rate if suitable.

 Keep the cut of root apex and bone cavity as small as possible. The use of retrograde instruments requires much less space than when using conventional files or instruments.

 Perform a full retrograde canal preparation prior to isthmus preparation. Insert the instrument in one of the two canals where the isthmus begins, then gently move it along the isthmus to create a small groove.

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The cavities must be prepared before using the Piezon instruments. Previous restorations must be completely removed. Use a standard excavator for the excavation of tooth decay cavities or caries.

#### Instruments SD and SM

Partially diamond-coated instruments for preparing distal and mesial cavities without damaging the adjacent tooth.

#### Instrument SB

Diamond-coated instrument for opening pits and small caries.

#### Instruments SBd and SBm

Partially diamond-coated instruments for beveling and finishing mesial and distal cavity margins.

#### Instrument VE

Diamond-coated instrument for preparing the tooth for placement of veneers.

#### Instrument PF

Diamond-coated instrument for cleaning and enlarging fissures before sealing.

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## Amalgams and cementation

Unlike other Piezon instruments, the SP instrument must be used with no irrigation in "Dry Work" mode. If this mode is not available, select the lowest available irrigation flow rate.

 These instruments must be activated intermittently because their tip heats up very rapidly. Thus the activation of the instrument must not exceed 1 minute for a maximum use time of 10 minutes.

### Instrument SP

Instrument used for the cementation of ceramic inlays, onlays and veneers with highly thixotropic dual-curing composite cements.

The ultrasound oscillations are transmitted through the inlay or onlay to the composite cement. The thixotropic properties and the ultrasound effect liquefy the cement for a short time to spread it evenly.

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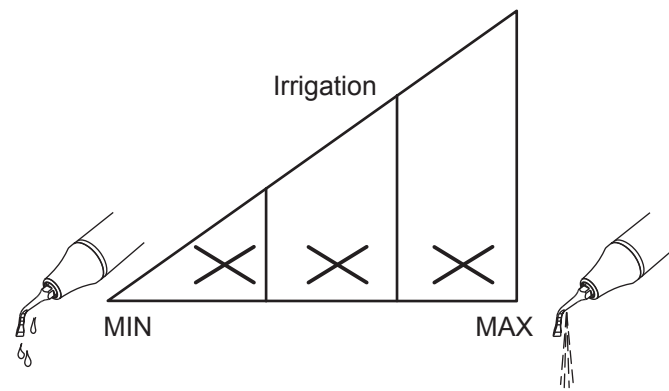
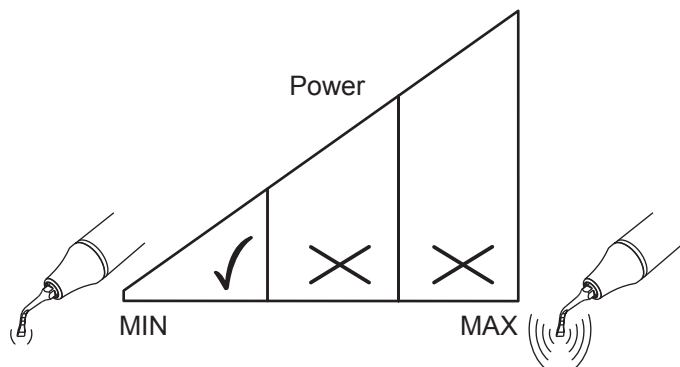
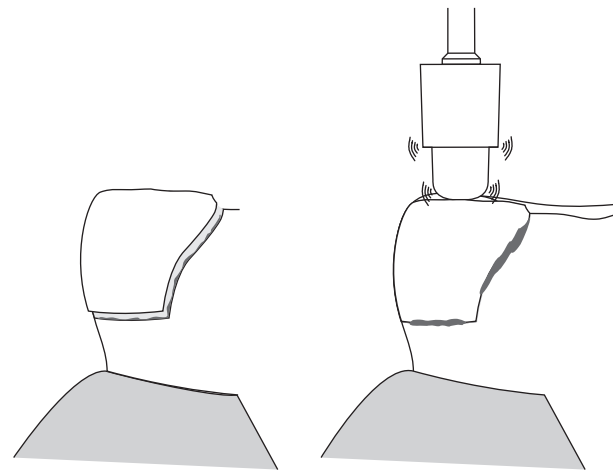
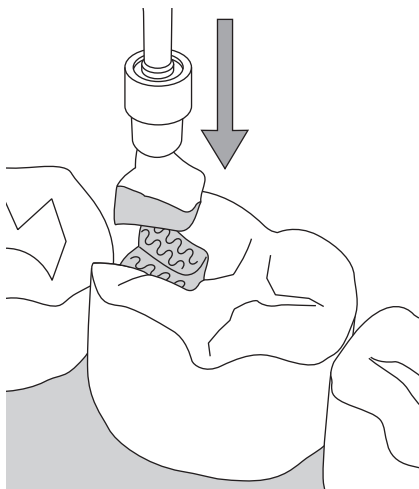
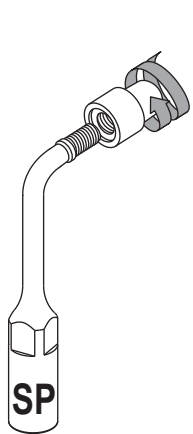
RESTO

## Cementation instrument SP

Instrument for placing and cementing inlays, onlays and veneers with a highly thixotropic dual-curing composite cement.

Use a low power setting and no irrigation.

Screw the cap onto the instrument and tighten it finger tight. Position the inlay or onlay to its final seating position without exerting pressure on the instrument. The composite cement returns to solid state as soon as the ultrasound activation is stopped.

The caps wear during use and must be replaced once the threads are not tight anymore, or when the cap surface shows wear.

We recommend to use a rubber dam to prevent the patient from accidentally swallowing or inhaling parts or debris. When a rubber dam cannot be used, instruct the patient to breathe through the nose.

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## PERIODONTICS

 Avoid using the HVE (high volume evacuation) to ensure sufficient cooling of the tooth and the instrument. The instrument heats up quickly if the irrigation flow rate is insufficient.

### Instruments PL1 and PL2

The instruments PL1 (left-curved) and PL2 (right-curved) are used for periodontal debridement with difficult access to the approximal surfaces.

### Instrument PL3

Instrument for the irrigation and disinfection of periodontal pockets with an antimicrobial solution.

### Instruments PL4 and PL5

The instruments PL4 (left-curved) and PL5 (right-curved) with a ball-shaped tip are used for debridement in root furcations and concavities.

### Instrument HPL3 (coarse)

Diamond-coated instrument (70 µm roughness) for in-depth root debridement using direct vision (flap technique), as well as for smoothing restoration overhang and expanding root furcations.

### Instrument DPL3 (smooth)

Diamond-coated instrument (15 µm roughness) for polishing surfaces first cleaned and shaped using the instrument HPL3 above.

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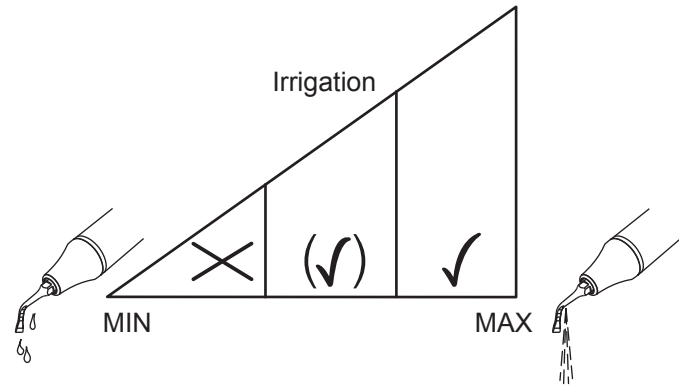
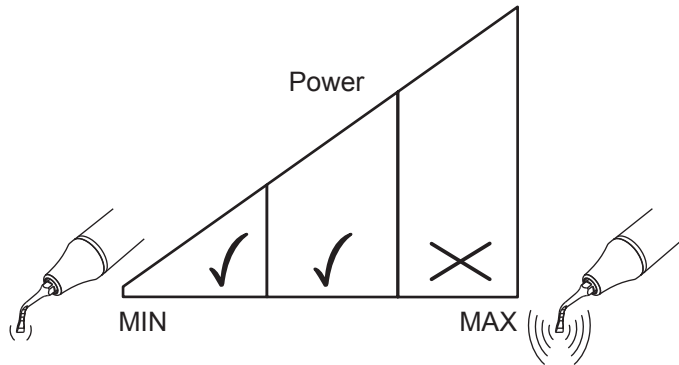
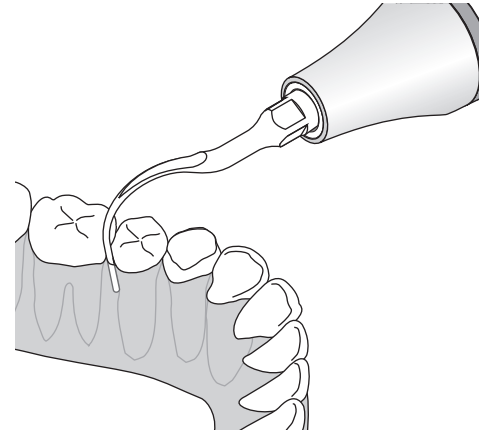
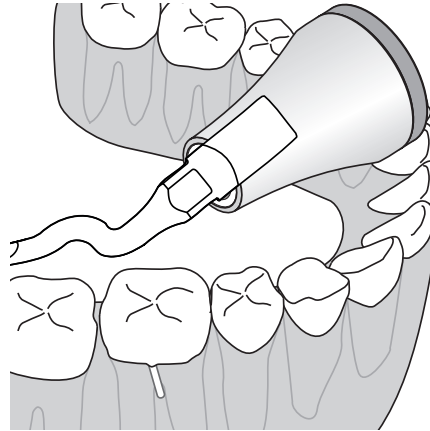
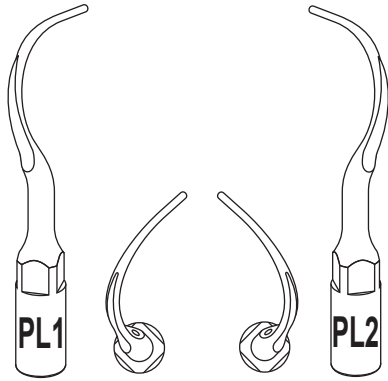
## Perio line instruments PL1 and PL2

Curved instruments for periodontal debridement, especially for difficult to reach approximal areas.

Use a low power setting or medium power for hard concretions. Never exceed a medium power setting.

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For an efficient full-mouth treatment only use the **outside** lateral part of the instrument PL1 and PL2 as shown on page 72.

Apply only a light lateral pressure.

 Do not direct the instrument straight to the enamel surface. Never point the tip of the instrument to the tooth surface.

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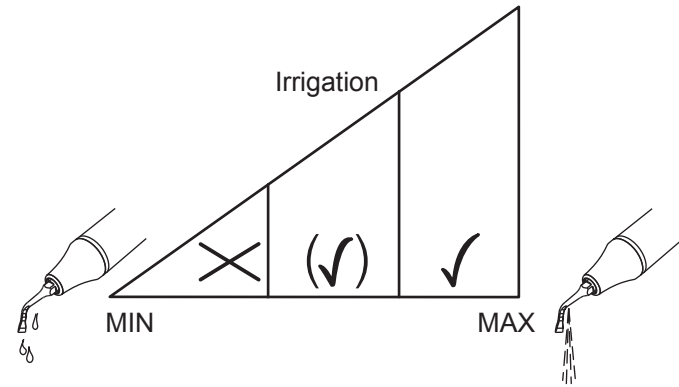
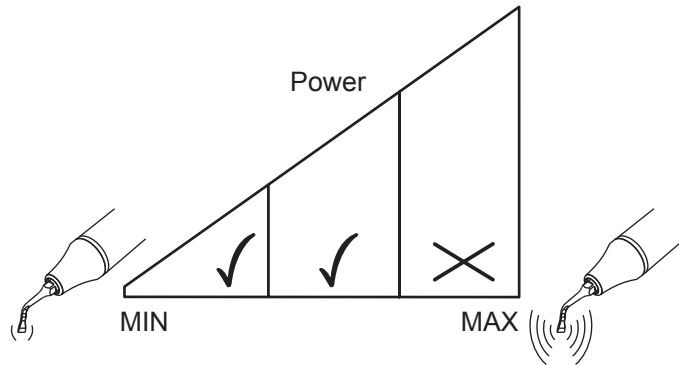
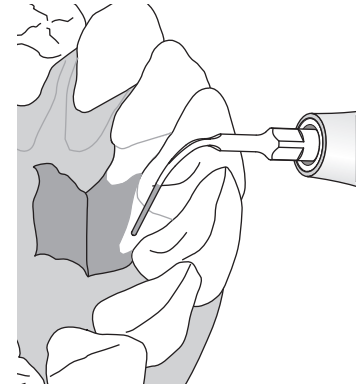
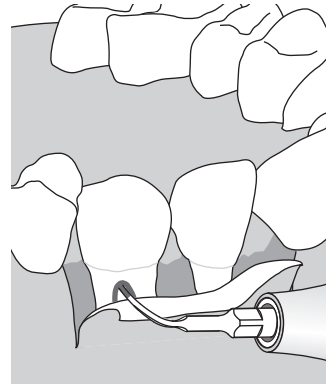
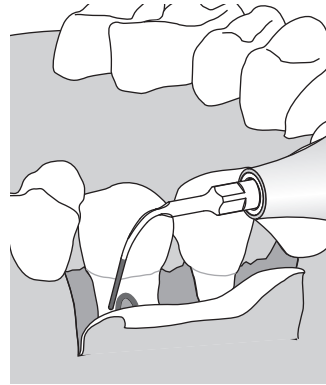
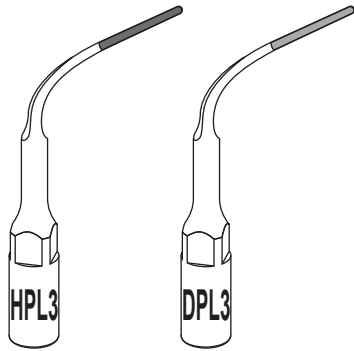
## Instruments perio HPL3 and DPL3

### HPL3: 70 µm roughness

Diamond-coated instrument for in-depth root debridement under direct vision using a flap technique. Also suitable for smoothing restoration margins and for expanding root furcations.

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### DPL3: 15 µm roughness

Diamond-coated instrument for polishing surfaces previously cleaned and shaped using the instrument HPL3 above.

Use a low to medium power setting and a high irrigation flow rate.

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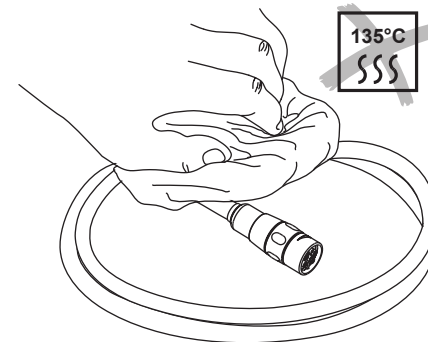
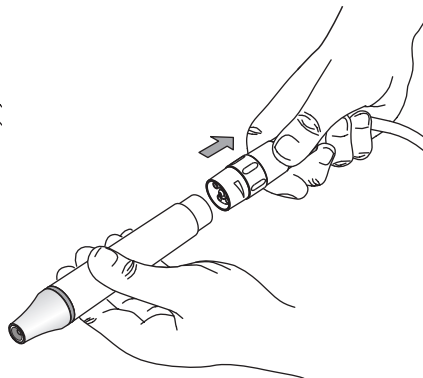
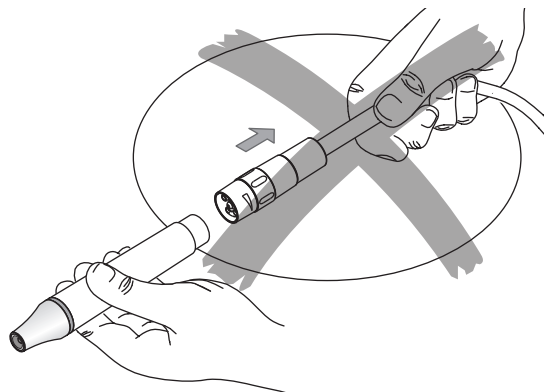
## CLEANING, DISINFECTING AND STERILIZING

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 Follow present-day regulations enforced in the country about reprocessing.

 Do not immerse the handpiece or the cords in a disinfectant solution as this may damage them.

 Use only an alcohol-based, commercially available (ethanol, isopropanol), colorless disinfectant. The use of scouring powder or abrasive sponge will damage the surfaces.

Place the cleaned and disinfected components in sterilization bags or in the EMS SystemBox (stains may appear on the SystemBox surface after sterilizing).

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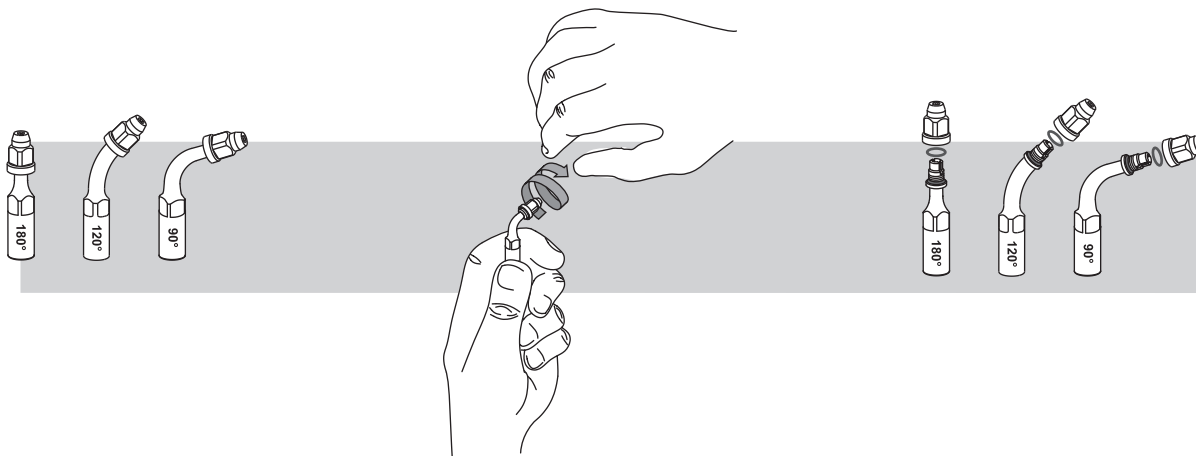
## File holder disassembly and cleaning

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 Remove the tightening nut and the O-ring (when equipped) to clean each component under running water before disinfection.

 Please follow the recommendations of the Reprocessing Instructions manual delivered with your product regarding the procedures for cleaning, disinfecting, sterilizing and packing of the components.

 Regularly check that the O-ring on the file holder is not damaged.

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
## WEAR AND MAINTENANCE
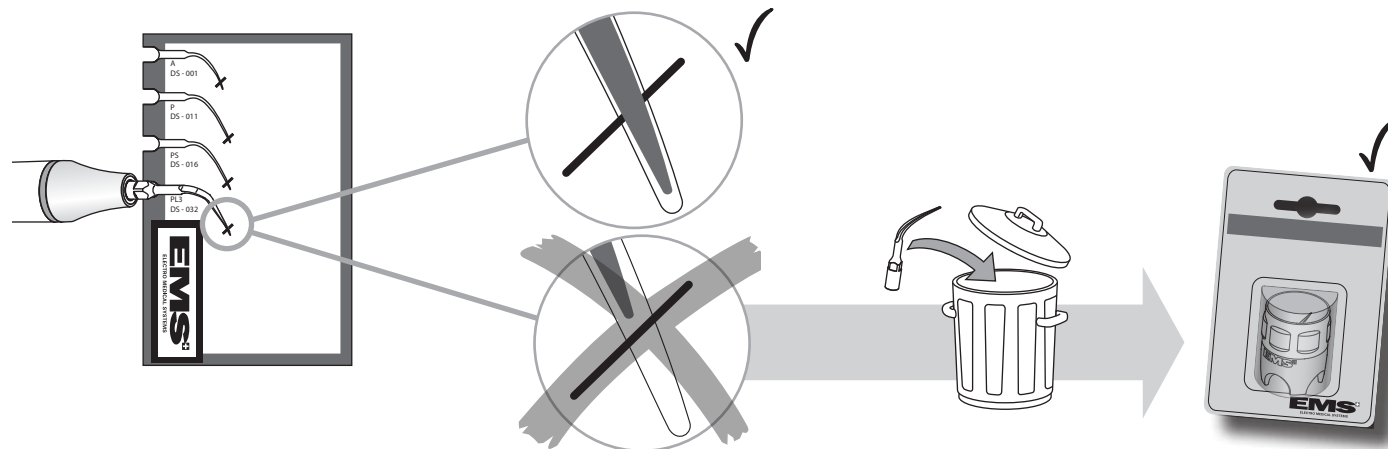
## USURE ET ENTRETIEN

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The EMS Piezon instruments must be checked regularly using the EMS "Tip Card". Ultrasound instruments wear during use and become shorter. Worn instruments are ineffective and are a cause of discomfort to the patient.

 Instruments showing wear on the diamond coating must be replaced.

 For precautionary reasons, do not exceed the validated lifespan of the components (see the Technical Data chapter).

 Always use EMS original parts. Using non original components may damage the equipment, and practitioner or patient may be injured.

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## SAFETY PRECAUTIONS

EMS and the distributor of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 Only use this product for the intended indications. Carefully read these operating instructions before using the product. This also applies to any product used with this system. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged.

 Do not use this product in the presence of inflammable anaesthetics or gases.

 This product must only be used by trained and qualified personnel.

 In rare cases the high frequency oscillations can break activated instruments or files, particularly when not manipulated properly. Place a rubber dam before treatment to prevent the patient from swallowing or inhaling parts of a fractured instrument, or a loose instrument. If a rubber dam cannot be placed, instruct the patient to breathe through the nose during treatment.

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 To avoid heat damage on the tooth, never work dry unless the instrument is specifically designed and indicated as such. The tip of the activated instrument heats up immediately when used dry. The instrument must not be activated during more than 10% of a maximum working time of 10 minutes with the "Dry Work" function. An incorrect use may also cause a deterioration of the handpiece.

 Make sure to always sterilize handpieces and instruments before use. Non-sterile handpieces and instruments may cause bacterial or viral infections. Please follow the recommendations of the "Reprocessing Instructions" manual delivered with your product regarding the procedures for cleaning, disinfecting and sterilizing of the components.

 Always examine your Piezon system for damage before commencing treatment. Damaged accessories or a damaged product must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Listen for a noise change when the instrument is activated to detect a possible loosening of the instrument in the handpiece. This also applies to file holders.

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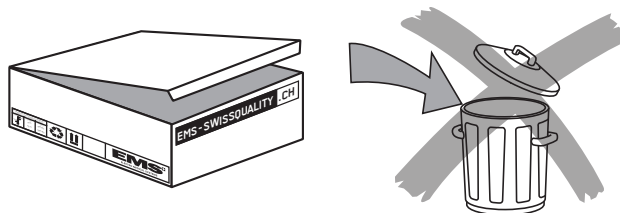
## STORING THE PRODUCT

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 Keep the original packaging until the product is to be disposed of permanently. It can be used for shipping or storing at any time.

Should you wish to store the product for an extended period of time:

- Proceed as described in the "Cleaning, disinfecting and sterilizing" chapter,
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data" section.

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## PRODUCT DISPOSAL

The product must not be discarded in domestic household waste. Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

Waste Electrical and Electronic Equipment belonging to customers located in the European Union may be shipped to EMS for recycling in accordance to the WEEE regulations. The costs of recycling, exclusive of shipping fees, are covered by EMS.

## WARRANTY

Damage due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

## ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

 For safety reasons this product may only be used with genuine EMS accessories. The user assumes all risks of using this product with non-genuine accessories or accessories which are not compliant with the operating instructions.

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## EMS CUSTOMER SERVICE

Please contact your dealer or EMS repair center for any servicing or repairs.

EMS decline responsibility for the safety of the device and declare the warranty null and void if servicing or repairs are carried out by an unauthorized third party or if non-genuine spare parts are used.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

 Before dispatching your product including all accessories, please clean, disinfect and sterilize them as described in the operating instructions.

 Risk of transmitting micro-organisms! To protect the personnel of the repair center and for safety reasons during transport and shipment, the product and the accessories returned to the factory for repair or revision must be cleaned, disinfected and sterilized in accordance with the operating manual. Repair can be refused for product or accessories received in a contaminated condition.

When sending your product directly to an approved EMS repair center, please include the name of the dealer to simplify processing.

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## TECHNICAL DATA

### Description

Manufacturer	EMS Electro Medical Systems SA CH-1260 Nyon, Switzerland
Model	Piezon® Instruments and Systems
Classification 93/42 EEC	Class IIa
Operating conditions	+10 °C to +40 °C 30% to 75% relative humidity
Storage and transport conditions	-10 °C to +40 °C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure
Validated life span: - Instruments and CombiTorques - SystemBoxes - Handpieces	100 sterilization cycles 2000 sterilization cycles 1000 sterilization cycles (in autoclave at 135°C)

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## TROUBLESHOOTING

Symptom	Solution
No or low irrigation flow	<ul style="list-style-type: none"><li>• Check if the instrument is blocked. Blow compressed air through the instrument to unblock it</li><li>• Use another handpiece to check if the handpiece is blocked</li><li>• If the obstruction cannot be removed, the handpiece must be sent to an approved EMS repair center</li><li>• Please refer to the operation instructions of the Piezon unit for more details</li></ul>
No ultrasound oscillations	<ul style="list-style-type: none"><li>• Please refer to the operation instructions of the Piezon unit</li></ul>
Decreasing or reduced performance	<ul style="list-style-type: none"><li>• Follow the instructions found in the operation instructions of the Piezon unit</li><li>• Make sure that the instrument is properly positioned, tighten it again with the CombiTorque</li><li>• Check the instrument for wear and replace it if needed</li><li>• Try to use the handpiece with another instrument</li><li>• Send the handpiece and the instrument to an approved EMS repair center</li></ul>
Breakage of a file or instrument, possibly inside the cavity or the root canal	<ul style="list-style-type: none"><li>• Make sure that every broken part is located</li><li>• Compare total length of the broken parts with the length of a new file or instrument to make sure that all broken parts are present</li><li>• Try to remove broken parts inside the root canal by activating maximum irrigation (no ultrasound) on a file to attempt to force the debris out</li><li>• Refer to instructions and use the RT2, RT3 and RT4 instruments that are specially designed for this application</li></ul>
Diamond-coated instruments become less effective	<ul style="list-style-type: none"><li>• Visually check the condition of the diamond coating and replace the instrument if needed</li></ul>

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TRADING (SHANGHAI) CO., LTD.