

# JET XR

## Radiopaque Acrylic

Caution: Federal law restricts this device to sale by or on the order of a dentist.

### Product Identification

Auto-polymerizing acrylic polymer with radiopaque properties

Shadow	The equivalent of 1.0 mm of aluminum
Opaque	The equivalent of 1.8 mm of aluminum

### Package Contents

REF 4310-20	Net Wt. 45gm, Shadow
REF 4310-40	Net Wt. 45gm, Opaque
REF 4320-20	Net Wt. 100gm, Shadow
REF 4320-40	Net Wt. 100gm, Opaque

### Intended Purpose

Jet XR Radiopaque Acrylic Resin is intended for the fabrication of methacrylate-based acrylic appliances and used wherever a radiopaque acrylic dental device is useful in establishing an anatomical location or site, such as a provisional implant stent.

### Storage Conditions/Instructions

Close container tightly after use. Store in original container in a cool, dry, well-ventilated location, away from sources of ignition and light.

### Operating/Handling Instructions

Do not ingest. Avoid contact with skin, eyes and clothing. Avoid breathing dust. See Material Safety Data Sheet for complete information. MSDS is available at [www.langdental.com](http://www.langdental.com) or by contacting manufacturer.

### Contraindications

Jet XR is contraindicated in patients with allergies to methyl methacrylate.

## Other Materials Needed

Jet Liquid

## Instructions for Use

1. Shake Jet Liquid before use.
2. Measuring by volume, mix 3 parts Jet XR powder with 1 part Jet Acrylic liquid. (For a less viscous mixture, add more liquid.)
3. Use mixture for desired application. Setting time is 12-15 minutes at 72°-78°F/22°-26°C (longer if additional liquid was added.)
4. Desired radiopacity is achieved as noted above.

## Disposal

Dispose of unused material in accordance with local, state and national legislation.

## Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark<sup>®</sup> Europe  
BP 2332  
38033 Grenoble Cedex 2  
France



Lang Dental Mfg. Co., Inc.  
Wheeling, IL 60090-0969 USA  
[www.LangDental.com](http://www.LangDental.com)  
Made in USA  
Ph 847-215-6622  
Fax 847-215-6678

The word "LANG" in a stylized, italicized, serif font, with a horizontal line above and below the letters. A thin line with an arrowhead points from the bottom left towards the "L".

LANG



LDD 43 R.0